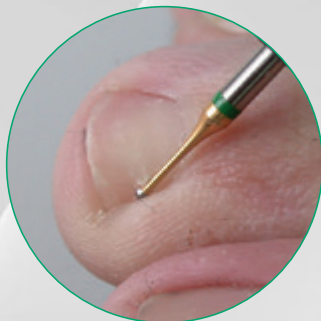
Push[®]

There is no substitute for quality

Podoinfo

Rotierende Instrumente in Praxis und Ausbildung
Rotary instruments for practice and education

Christian Rothe

Gert Busch

There is no substitute for quality

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What is a medical device?

Medical Devices Act

Basis for medical devices (CE marking)

Based on EC directive 93/42/EEC – Medical Devices Directive, or MDD – the Medizinproduktegesetz (Medical Devices Act) was passed into law in Germany in 1994. This law and its associated regulations remained in force until 25 May 2021, when they were replaced by European Regulation (EU)2017/745 of the European Parliament and of the Council, known as the MDR.

Busch & CO. GmbH & Co. KG has satisfied the requirements for a compliant quality management system according to the DIN EN ISO 13485 standard since 1998.

The successful implementation of the system is confirmed regularly as part of an annual audit.

By virtue of this procedure, Busch & CO. GmbH & Co. KG is legally obliged and entitled to use the CE mark for its medical devices.

All instruments manufactured by BUSCH for medical chiropody / podiatry are medical devices which are identifiable as such from the CE marking and the four-digit number 0124 on the label of each individual package that provide proof of conformity.

In podiatry, brushes and polishers are considered to be cosmetic products, so they do not bear the CE mark.

Who must use medical devices?

In general, users can be divided into two groups:

1. In this context, chiropodists / cosmeticians are, by definition, only permitted to work on, treat and dress healthy tissue.
This includes cutting healthy nails, cleaning and outlining healthy nails, removing excess callus, skin care and cosmetic artificial nails.
The use of medical devices is not mandatory for this occupational group.
However, we recommend using the higher grade medical devices if only to avoid risk to yourself as the treatment professional and to your patients.
2. Podiatrists / medical chiropodists carry out healthcare activities (including on pathologically altered tissue) and only earn this entitlement after receiving the corresponding training at a state-approved training facility (school of podiatry).
Such persons are subject to the provisions of the Medizinprodukte-Betreiberverordnung (Ordinance on the Installation, Operation and Use of Medical Devices).
In order to ensure compliance with the Medizinprodukte-Betreiberverordnung and for the safety of patients and users, such activities are conditional upon the use of medical devices.

How should medical devices be treated in podiatric practice?

When authorised for use, medical devices are associated with intended purposes.

For example, the intended purpose of podiatric cutters is to remove excess callus or abrade nail material affected by fungal infection.

Since these actions are therapeutic procedures involving pathologically altered tissue and it is therefore not possible to avoid injury and contact with blood, any such medical device must be sterilisable.

This is the only way to guarantee that hygienic reprocessing (sterilisation) of the medical devices used will effectively prevent cross-contamination between one patient and the next, and prevent pathologically altered tissue from becoming further infected / contaminated by unhygienic instruments. Depending on the invasivity of the therapy performed and the design of the medical device itself, used medical devices are classified as „semi-critical A/B“ or „critical A/B“ and must be sterilised using correspondingly defined reprocessing procedures.

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All instruments are packaged appropriately to ensure cleanliness and protect them from damage.

Welche Informationen beinhaltet das Produktetikett?
What kind of information can you find on the product label?

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Yes, because all of our instruments
are manufactured according to
the **DIN EN ISO 1797-1**
standard.
This standard is also applicable
to all manufacturers of podi-
atric drives. Consequently, our
instruments are guaranteed to fit
in your handpiece perfectly, for
both dry and wet techniques.

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The standardised dimensions are:

A - shank diameter Ø 2.334 to 2.350 mm
B - minimum clamping length 30 mm

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Other important features according to application field:

D - working part diameter e.g., 060 = 6.0 mm
L - working part length x mm
M - working part material e.g., carbide

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In general, instruments are
distinguished according to the
type and material of the working
part (grit or blade).

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**Instruments with
defined grains**

- hybrid instruments
- diamond instruments
- ceramic abrasives
- polishers

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**Instruments with
defined cutting edge**

- carbide cutter
- ceramic cutter
- steel cutter

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- fine, sharp blades, similar to scalpels
- ideal for treatments on skin, calluses and nail walls

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Abrasives

- for smoothing nail surfaces, calluses and artificial nails
- Surface renewal during use. The initial shape is diminished by wear, but „fresh“ grits are always available.

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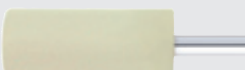
Polishers

- similar to the abrasive tools, but the abrasive grit is much more elastic and more flexible bonded
- polishing of nail surfaces or artificial nails as part of the nail prosthesis.

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The combination of abrasive grits - diamond and ceramic - with the proven TWISTER design make the Hybrid TWISTER a high-tech product for modern podiatry.

Despite the essentially finer hybrid grit, it delivers significantly greater abrasion performance in both dry and wet techniques.

At the same time, it leaves behind a smoother surface.

Treatment with the Hybrid TWISTERS is particularly gentle, even when treating diabetic feet and patients with parchment skin. The instruments also have impressively long service lives.

- ● **super grobe Hybridkörnung** Abräumen schwerer Verhornungen
super coarse grit removal of severe callus

					
BUSCH	HT5880	HT5880		HT5854R	HT5854R
ISO Ø	105	125		105	125

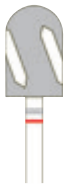
- ● **grobe Hybridkörnung** Abtrag dicker Hornschwielen
coarse grit removal of thick callus



						
BUSCH	HT6880	HT6880	HT6880		HT6854R	HT6854R
ISO Ø	080	100	120		100	120

- ● **mittlere Hybridkörnung** Entfernung starker Verhornungen
medium grit removal of hard callus

						
BUSCH	HT880	HT880	HT880		HT854R	HT854R
ISO Ø	075	095	115		095	115

- ● **feine Hybridkörnung** Entfernung beginnender Verhornungen
fine grit removal of incipient callus

						
BUSCH	HT8880	HT8880	HT8880		HT8854R	HT8854R
ISO Ø	075	095	115		095	115

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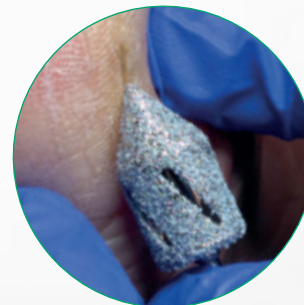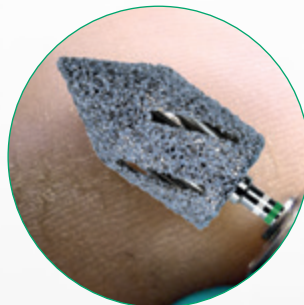Our expert for special cases

Remove large callus as well as rhagades, localised callus (e.g. on the tips of the toes) and corns with a single instrument.

The special feature of this Twister shape - the long, parallel head with a tapered tip - allows you to work both on larger areas and between the toes.

This makes the treatment process easier, and reduces both the need to change instruments and the time spent on device reprocessing.

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... the modern grit - diamond plus ceramic - is coated on a durable stainless steel cap.

Efficiency:

one HybridCap instead of hundreds

Sustainability:

instead of throwaway

Patient friendliness:

smoother surfaces in less time

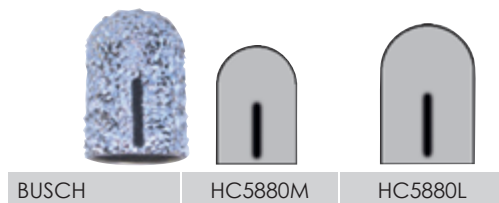
Hygiene:

HybridCap and Cap Support are disinfected and sterilizable.


A CE-symbol medical product made by BUSCH

Cap and mandrel have to be disinfected and sterilized unmounted for hygienic and technical reasons.


- **super grobe Körnung**, Abräumen schwerer Verhornungen
super coarse grit, removal of severe callus


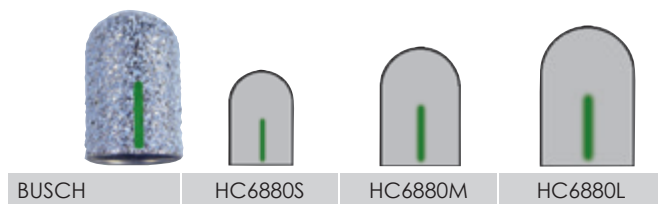
- **grobe Körnung**, Abtragen dicker Hornschwielen
coarse grit, removal of thick callus

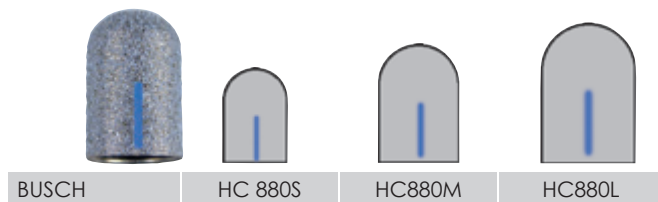- **mittlere Körnung**, Entfernen starker Verhornungen
medium grit, removal of hard callus

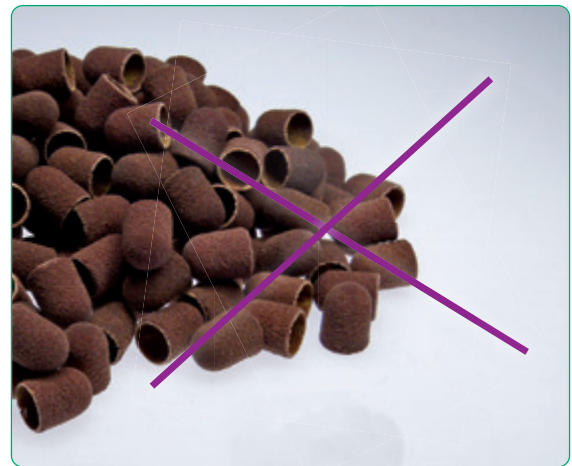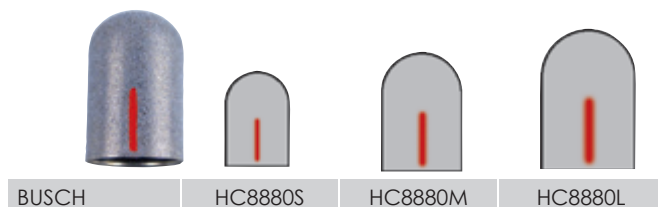- **feine Körnung**, Entfernen beginnender Verhornungen
fine grit, removal of incipient callus

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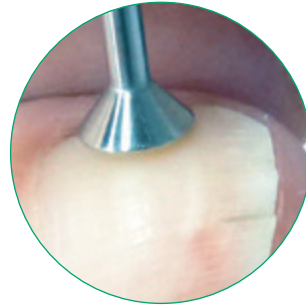
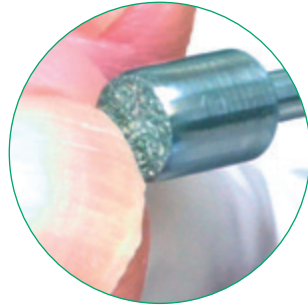
Extremely gentle nail shortening

Thanks to their intelligent design, the TOP Grip series are indispensable instruments for the targeted grinding of damaged, unnaturally thickened and deformed nails.

TOP Grip instruments enable nails to be shortened safely at the front and sides or the nail plate to be reduced down to the nail fold. The instruments have a smooth bevel with no diamond coating in the transition region to the diamond-coated front face.

With careful use, treatments of any kind can be completed without injury, which is essential, particularly when dealing with diabetics.

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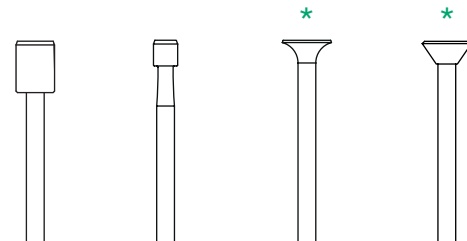
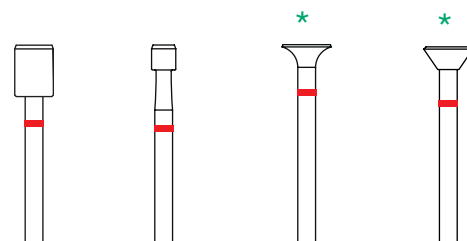
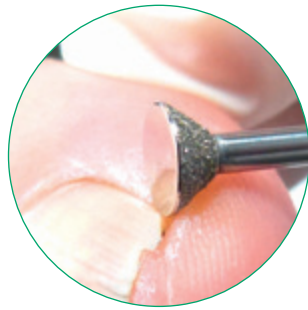
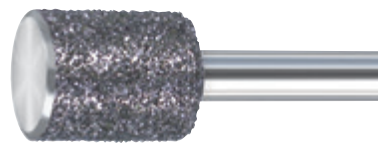
Treating nail wall regions

The SIDE Grip series are indispensable instruments for the targeted grinding of damaged or thickened nails, or critical areas of skin.

Only the lateral area of the working part is coated with diamonds, and the bevel in the transition region to the flat front face is smooth, with no diamond coating, so SIDE Grip instruments can be used to treat wall areas safely, with no unintended grinding contact.

With careful use, treatments of any kind can be completed without injury, which is essential, particularly when dealing with diabetics.

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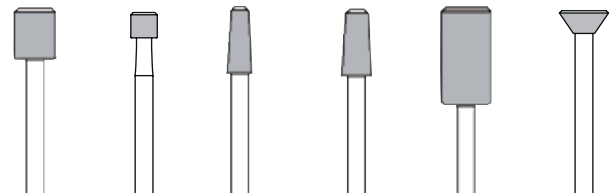
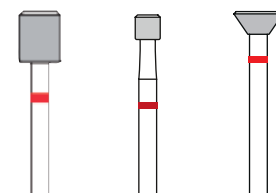
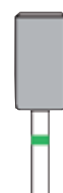
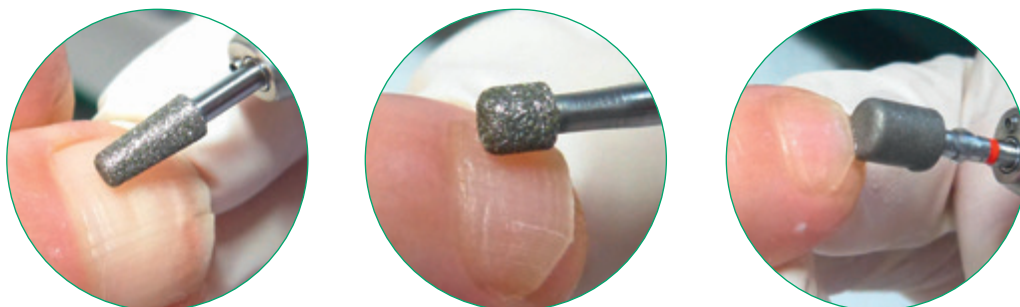
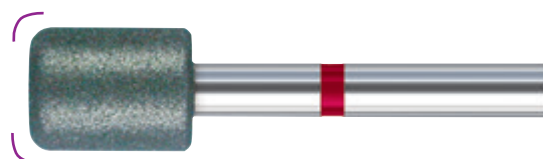
Gentle treatment of damaged nails

Rounded edges are the instantly recognisable feature of the KR series.


This enables extremely gentle smoothing of nails or abrasion of callus, particularly when dealing with vulnerable patients.

With careful use, treatments of any kind can be completed without injury, which is essential, particularly when dealing with diabetics.

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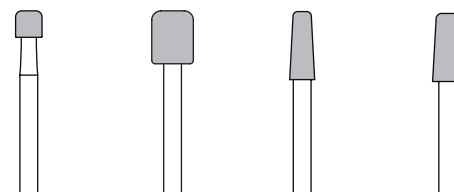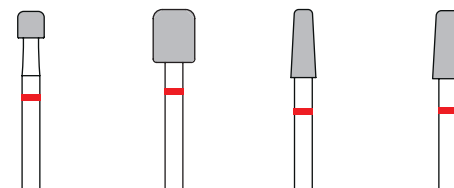
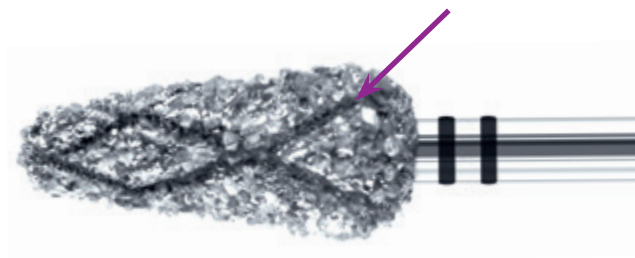
Abrasion of thick calluses and removal of hard calluses

The CD instruments with peripheral cooling groove technology for cool and pleasantly gentle work using dry and wet techniques.

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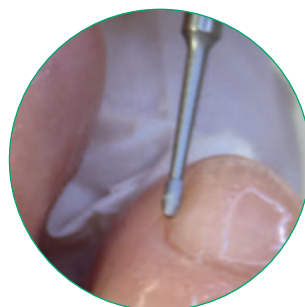
Gentle cleaning of the nail fold and removal of the cuticle in cases of heavier callus

In addition to the familiar classic ONY CLEAN the diamond-coated variant is particularly suitable for heavier cornification of the nail fold.

This variant is also particularly effective in areas calling for substantial abrasion in a closely defined location.

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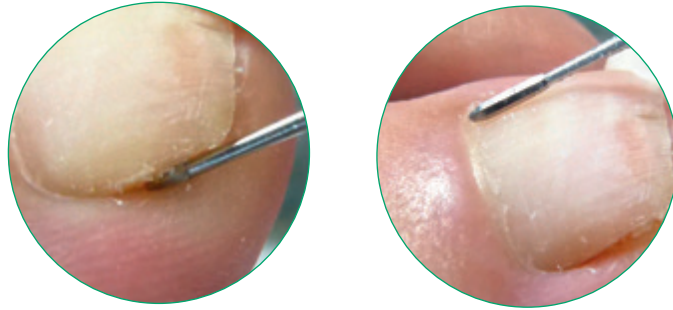
Gentle cleaning of the nail fold and removal of the cuticle

The special instrument made of stainless steel with the proven polygonal working part, without blades, which cleans the nail fold gently and leaves remarkably smooth nail surfaces.

The elongated shape of the working part is ideal for wide-area work.

Caution: Coarse diamond instruments or instruments with cuts used for this application will create unwanted striations on the nail.

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
with polygonal working part
without sharp cuts

BUSCH	407RS	406L-RS
ISO Ø	012	012

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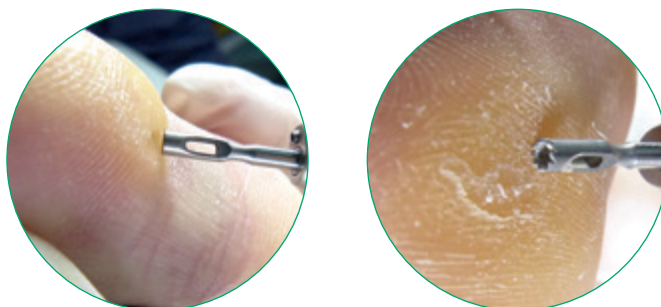Gentle removal
 of corns/clavi
 and callus, grinding
 of callus grooves

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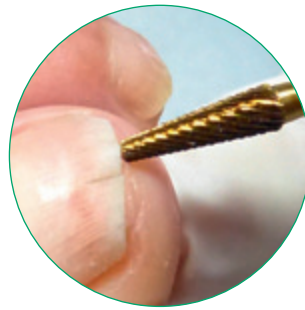
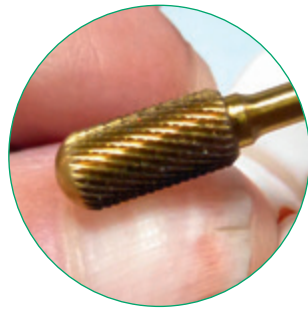
With their golden titanium nitride coating, they ensure fast, safe working in gel and acrylic in the nail prosthesis and on thickened nails

The particular advantages of this carbide cutter series are the highly effective SPEED cuts, which ensure rapid substance abrasion, and the smooth, domed working part tip without cuts to prevent injuries.

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Cutters for left-handed operators

We have introduced left-handed versions of the popular cutters into our range in order to offer left-handed users the same easy-to-use and convenient carbide cutters as right-handed users.

These carbide cutters can be identified from the labelling on the shank. They are marked with L at the end of the figure number.

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
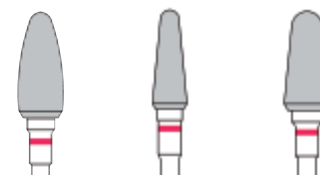
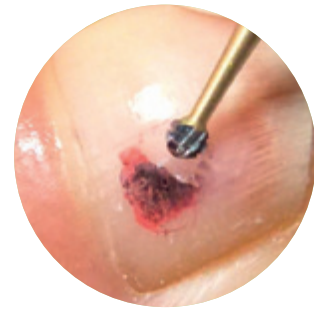
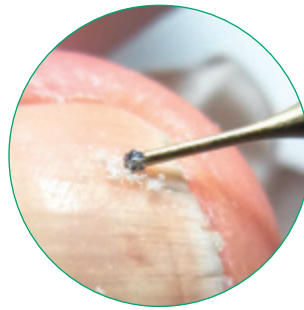
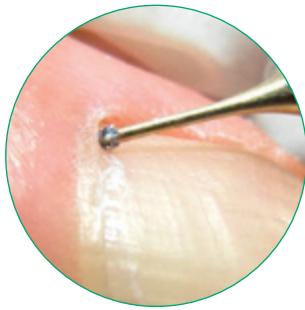
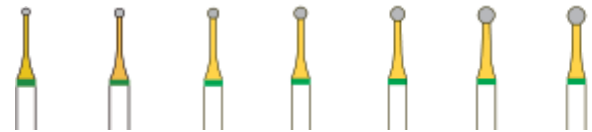
Soft removal of clavi and nail fungus as well as treatment of gryphotic nails and periungual callus

The slimline neck construction of this carbide cutter enables a better overview of the treatment site.

The special cut arrangement is instantly recognisable by the green shank ring and gold plated neck.

BUSCH	1SXM	1SXM	1SXM	1SXM	1SXM	1SXM	1SXM
ISO Ø	010	012	014	016	018	021	023

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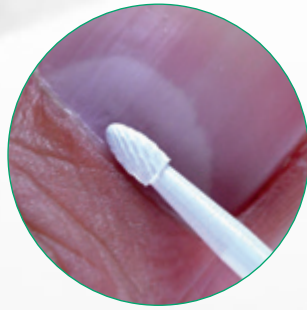All in white!

The filigree shapes entirely made of ceramic; for gentle treatment and precise shaping with an optimum overview.

This aesthetic material stands out through its excellent fracture toughness and hardness which, in combination with the proven BUSCH cuts, yields the typical attractive results.

The high-performance ceramic material used for KERA cutters is up to 25% respectively up to 60% lighter compared with carbide, thus permitting more gentle cutting – a tangible advantage for both patient and user when treating diseased natural nails as well as artificial nails.

Patients who are highly allergic to metals welcome the additional bio-compatibility. Naturally, KERA cutters are medical devices and suitable for all approved disinfection and sterilization methods.

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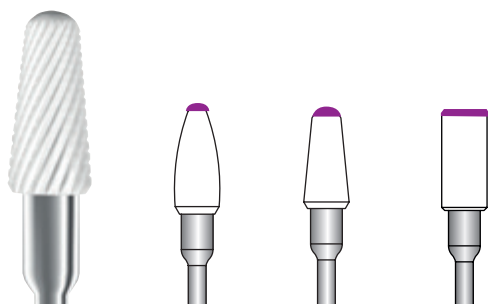
Special SPEED cut,
 with passive safety,
 recommended for diabetic patients

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From pre-polishing
to final polish, we offer
shapes for practically
all applications:

- polishing of the nail surface in nail prosthetics
- polishing of acrylates and nail prosthesis
- rounding and fine finishing of the nail edges
- smoothening sensitive skin areas
- smoothing of rough surfaces of natural nails and/or gels/acrylics

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smoothing, green

SiC, Silicone, Grit size coarse
bond hardness medium

BUSCH	9626G	9626G	9626G	9628G	9628G	9627G	9629G
ISO Ø	055	100	120	065	100	080	080

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polishing, anthrazite

SiC, Silicone, Grit size medium
bond hardness medium

BUSCH	9626	9626	9626	9628	9628	9627	9629
ISO Ø	055	100	120	065	100	080	080

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end-polishing, khaki

SiC, Silicone, Grit size fine
bond hardness medium

BUSCH	9622	9622	9622	9624	9624	9623	9625
ISO Ø	055	100	120	065	100	080	080

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high-grade corundum,
ceramic bonded

- for the removal of particles sticking to diamond instruments (grit size fine and coarser) at an instrument speed of 1.000-5.000 rpm.
- dampen before use

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coarse grit

- for dressing of polishers and abrasives

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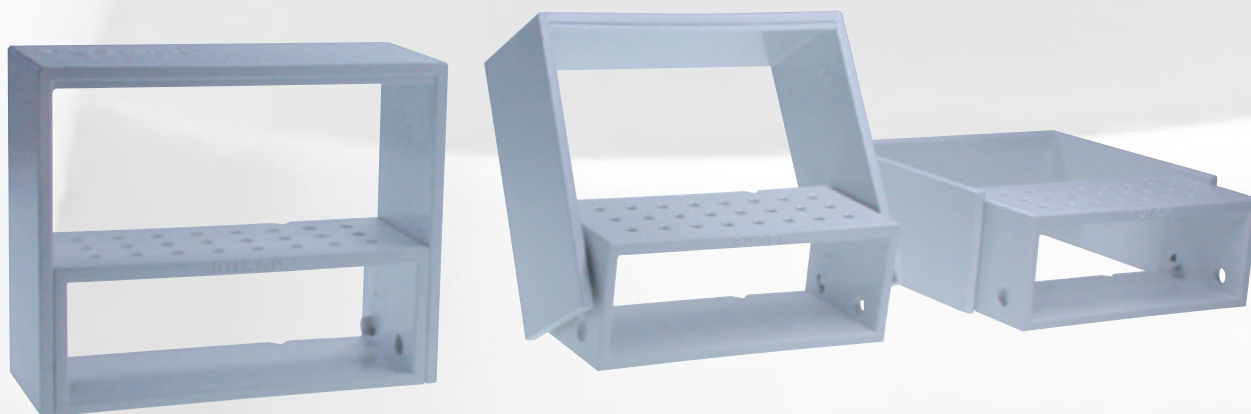
clever and hygienic

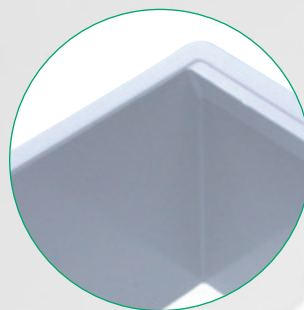
- robust overall design with improved stability
- rounded- off safety edges
- optimized opening and closing mechanism
- additional mounting options
- concentricity tester „control“
- for sterilization up to 180 °C
- for an individual and clear arrangement of instrument sets
- for the safekeeping of instruments before, during and after disinfection, cleaning and or sterilization
- for all usual methods of disinfection, cleaning and sterilization

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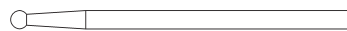
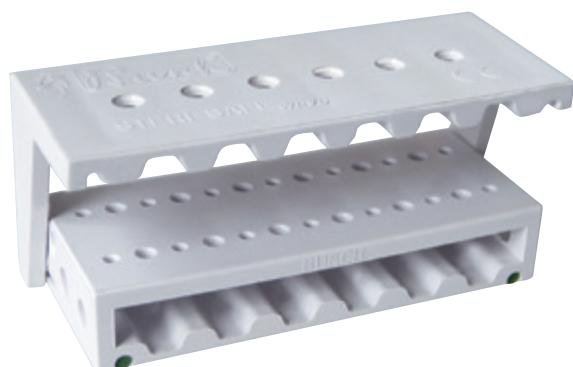
hygiene box PODO-LOCK
for the inclusion of a STERI-SAFEpro

Busch-No.: HPL 805

- securely lockable
- fillable with disinfectant for pre-cleaning
- dust-tight storage of disinfected/sterilized instrument sets
- not sterilizable

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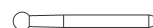
Busch-No.: 5773
dimensions: 76 x 30 x 33 mm
material: medical resin
for 14 instruments for the turbine shank FG Ø 1,60 mm

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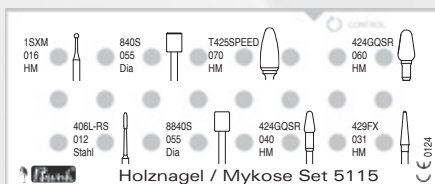
Basic set for precise, quick yet gentle treatment of mycotic / gryptic nails and for removing large quantities of nail substance

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All Busch sets are printed with the outline drawings, Busch no. and ISO size of the instruments that belong to the set.

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Thoroughly systematic working procedures, hygiene and organization are important for treating patients safely with rotary instruments.

systematic

To support your working procedures, we have designed instrument sets for specific applications. The configuration of these instrument sets is considered a recommendation only, and can be complemented as required. The sets are delivered in the hygiene box PODO-LOCK.

hygiene

The Steri-Safe_{pro} is a universal bur holder for storing and organizing burs as well as being perfect for disinfecting, cleaning and sterilizing specific instruments up to 180 °C. The instruments remain in the STERI-SAFE_{pro} throughout the hygiene procedure.

organization

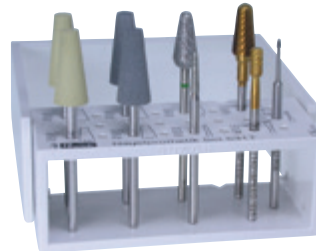
Practical instrument sets, enable the provision of hygienically prepared instruments, which can be transferred to the hygienic cycle after use, in preparation for the next treatment.

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
Stainless steel instruments for treating the nail fold, corns/clavi and for removing callus.

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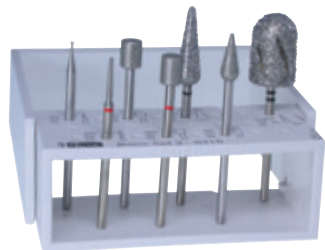
For specific treatment of diseased natural nails and artificial nail material in nail prosthetics.

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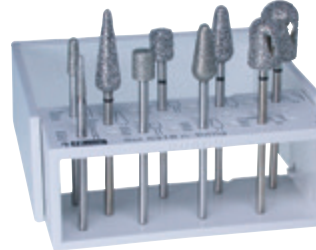
Diamond instruments in various grit sizes for gently grinding callus and nails.

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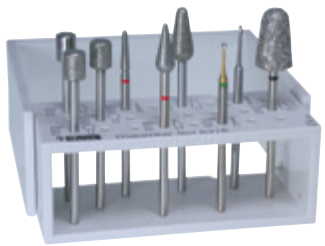According to the instruction and grinding principles published by Fritz Bittig. Usable in wet and dry techniques.

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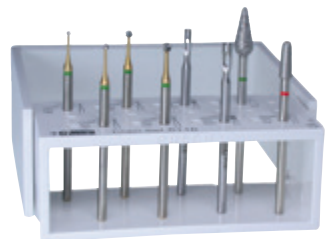
For minimum invasive treatment of the diabetic foot; rounded instruments with only partial diamond coating minimize the risk of injury during treatment.

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For grinding technique on skin and nails as indicated by Bittig.

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For specifically treating and removing corns.

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
The abovementioned sets are delivered in a hygiene box. The box can be closed firmly and provides protection from dust.

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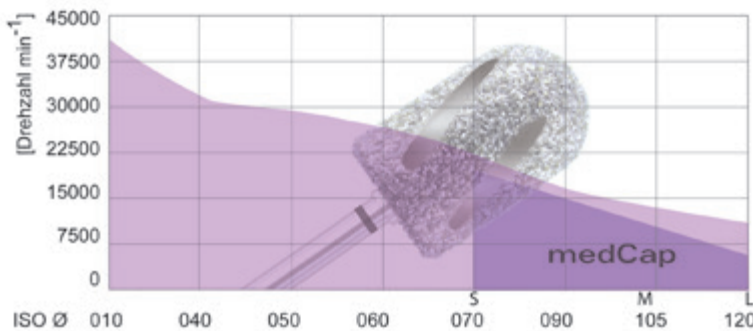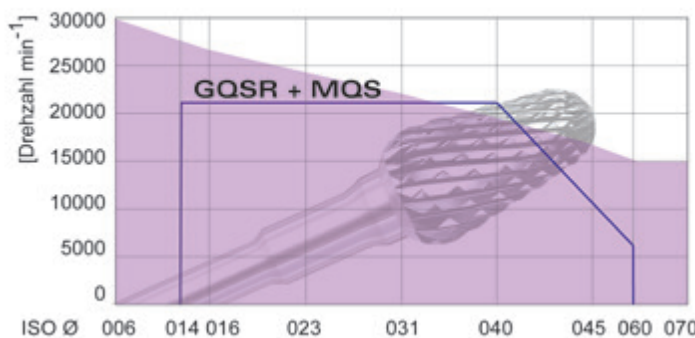
Hartmetall-Instrumente und KERA-Fräser
Carbide cutters and KERA cutters

max. admissible speed

The mentioned maximum admissible speed (rpm) indicated on each BUSCH pack is not allowed to be exceeded due to safety regulations.

speed ranges

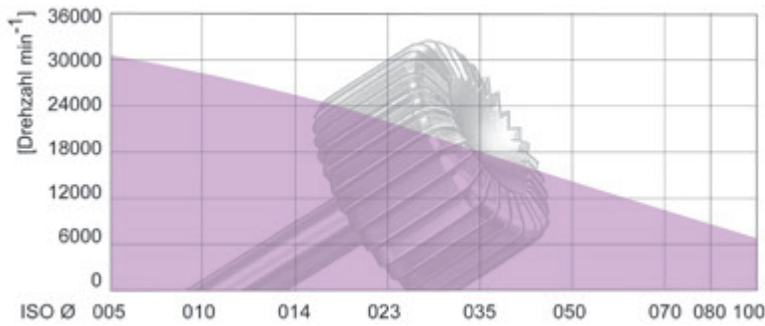
Keeping within the instrument-specific speed ranges leads to good treatment results (see diagram).

If the diameter of the working part exceeds that of the shank, powerful centrifugal forces may build up at higher speed which might bend the shank and/or fracture the instrument.

Rule of thumb: The larger the diameter of the instrument, the slower its speed.


Lower speeds should be selected in dry technique to keep any heat that might be generated to a minimum.

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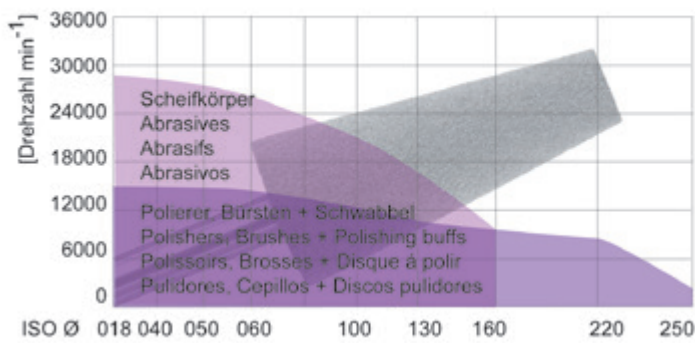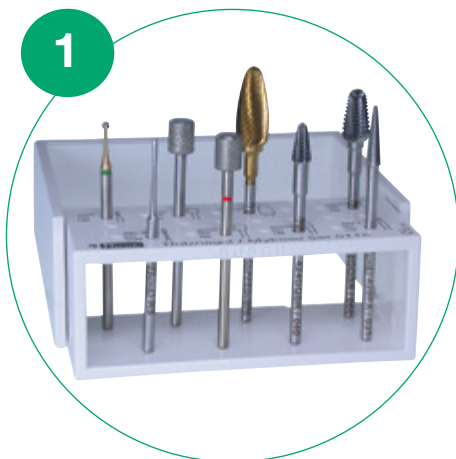
The use of the STERI-SAFEpro during hygienic reprocessing prevents your instruments from being damaged by knocking against each other.

Hygienically reprocessed instruments on the disinfectable and sterilisable STERI-SAFEpro instrument tray.

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Carbide cutter with speed cuts during treatment. (Example)

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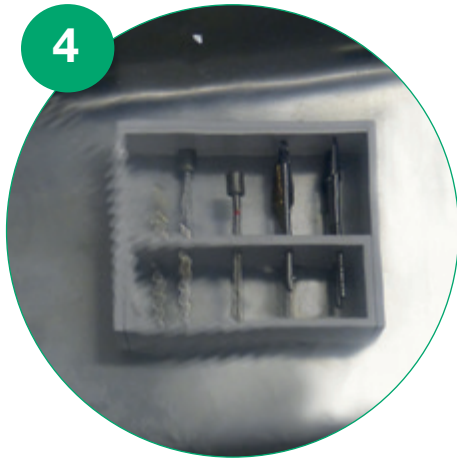
Instruments in the STERI-SAFEpro

As soon as the treatment has ended, remove any heavy soiling, place the instruments in the STERI-SAFEpro for preliminary disinfection, and store in the HPL box filled with disinfection fluid.

The mixing proportion of disinfection concentrates as well as the max. acting time as specified in the manufacturer's specifications must be observed meticulously.

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
Instruments in the STERI-SAFEpro during cleaning in the ultrasonic bath with cleaning additive. Then rinse with water and dry. If instruments are heavily soiled, it is advisable to carry out pre-cleaning by hand. For diamond instruments (grain size fine and coarser), use of the cleaning stone 599 is recommended.

Alternatively: Use of a cleaning/disinfection (RDG) unit, optionally with thermal disinfection.

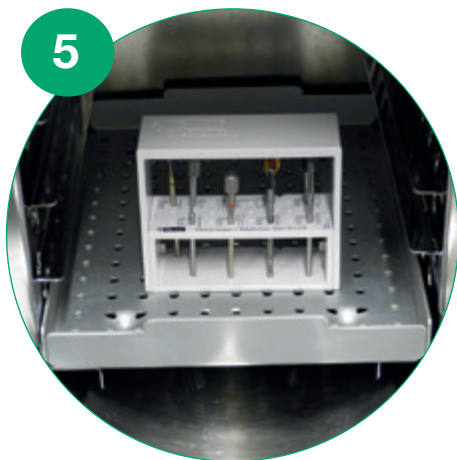Cleanliness check (with 6-8x magnifier), clean again if necessary!

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Instruments in the STERI-SAFEpro in the autoclave.

No sterilisation foil = Thermal disinfection

With sterilisation foil = Sterilisation

All instruments are designed to be sterilised with steam and hot air.

Exception: Polishers and brushes must not be subjected to steam and hot air sterilisation.

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After complete thermal disinfection, instruments are stored and protected from dust in the STERI-SAFEpro in storage box HPL 805.

After complete thermal sterilisation, instruments are stored accordingly in the STERI-SAFEpro in sterilisation foil.

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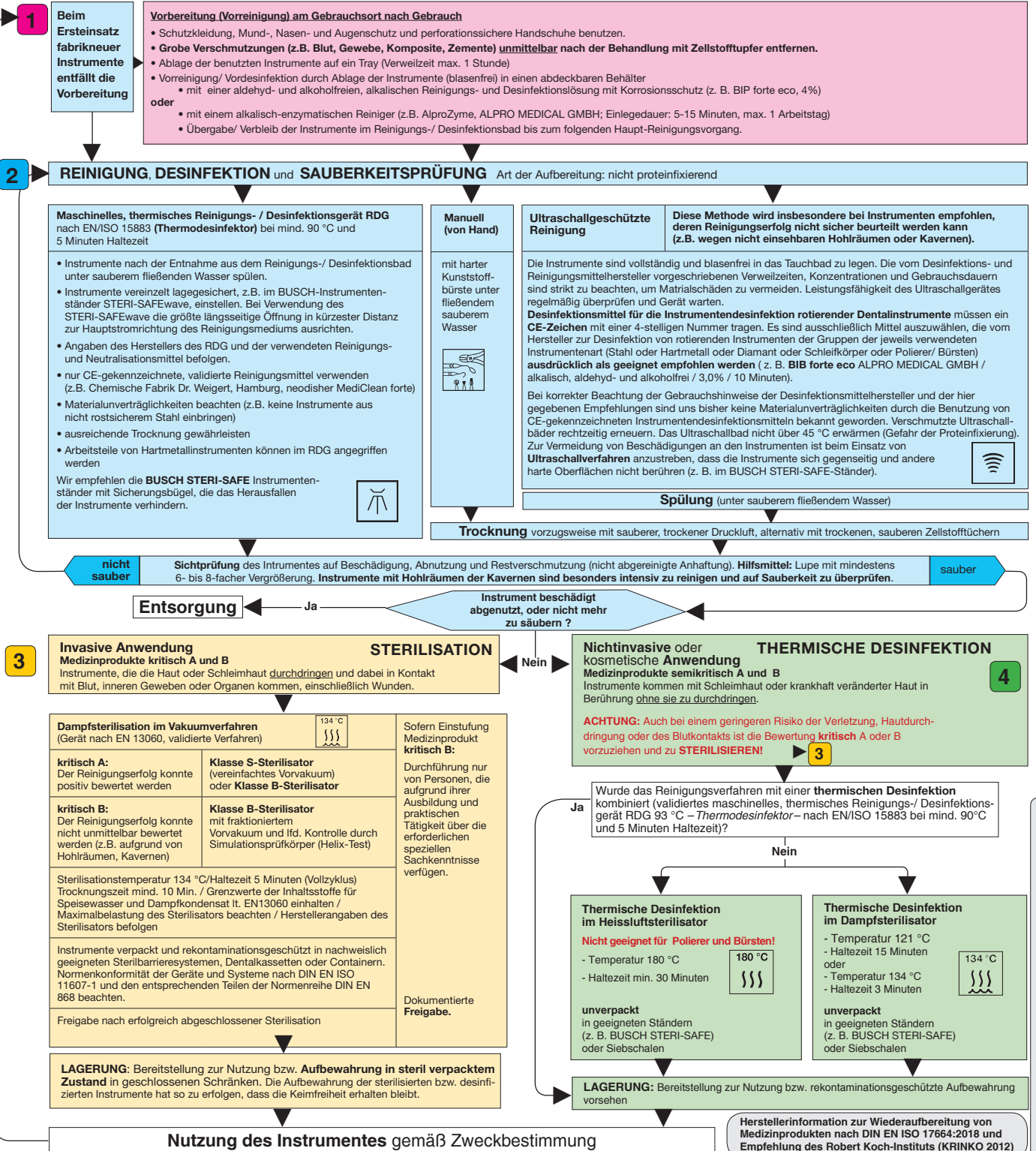
Safety recommendations

- Until they are first used, rotating instruments should be kept in their original packaging, and protected from dust and moisture, at room temperature.
- Always keep the packaging (also during the active usage period) so that the instruments are traceable if required.
- Always use fully functioning, correctly serviced and maintained as well as cleaned turbine drives as well as hand-held and angle-piece drives.
- Insert the instruments as deeply as possible. Check for firm seating.
- Use respiratory and/or eye protection and an extraction system.
- Before starting to work on the workpiece, bring the instruments up to operating speed and make sure that they are running concentrically.
- Observe the maximum speed (as indicated on all BUSCH packaging). Risk of overheating and injury.
- Make sure that the instruments don't get jammed or levered.
- Do not exceed a contact pressure of 0.3 to 2.0 N. (Risk of breakage and iatrogenic injuries in the working space and danger of heterotropic ossification after overheating/burning.)
- Only use the instruments in accordance with their intended use.
- Failure to comply with or adhere to the following hygiene recommendations can lead to transmission of pathogens.
- If possible, use the entire length of the working part in order to avoid point overheating, e.g. of the tips (resulting excessive mechanical stress and local overheating).
- Make sure that there is sufficient water cooling to avoid unwanted heat development (overheating). For the dental practice, this means a minimum cooling flow of 50 ml per minute ensure a minimum.
- FG instruments with a total length of more than 22 mm or a head diameter more than 2 mm might require additional cooling.
- Surgical instruments with a long shaft might require additional cooling.
- Bent or non-concentrically running instruments or instruments with damaged or worn working parts must be rejected and disposed of in order to avoid injury or overheating through friction caused by blunt instruments.

Hygiene recommendations

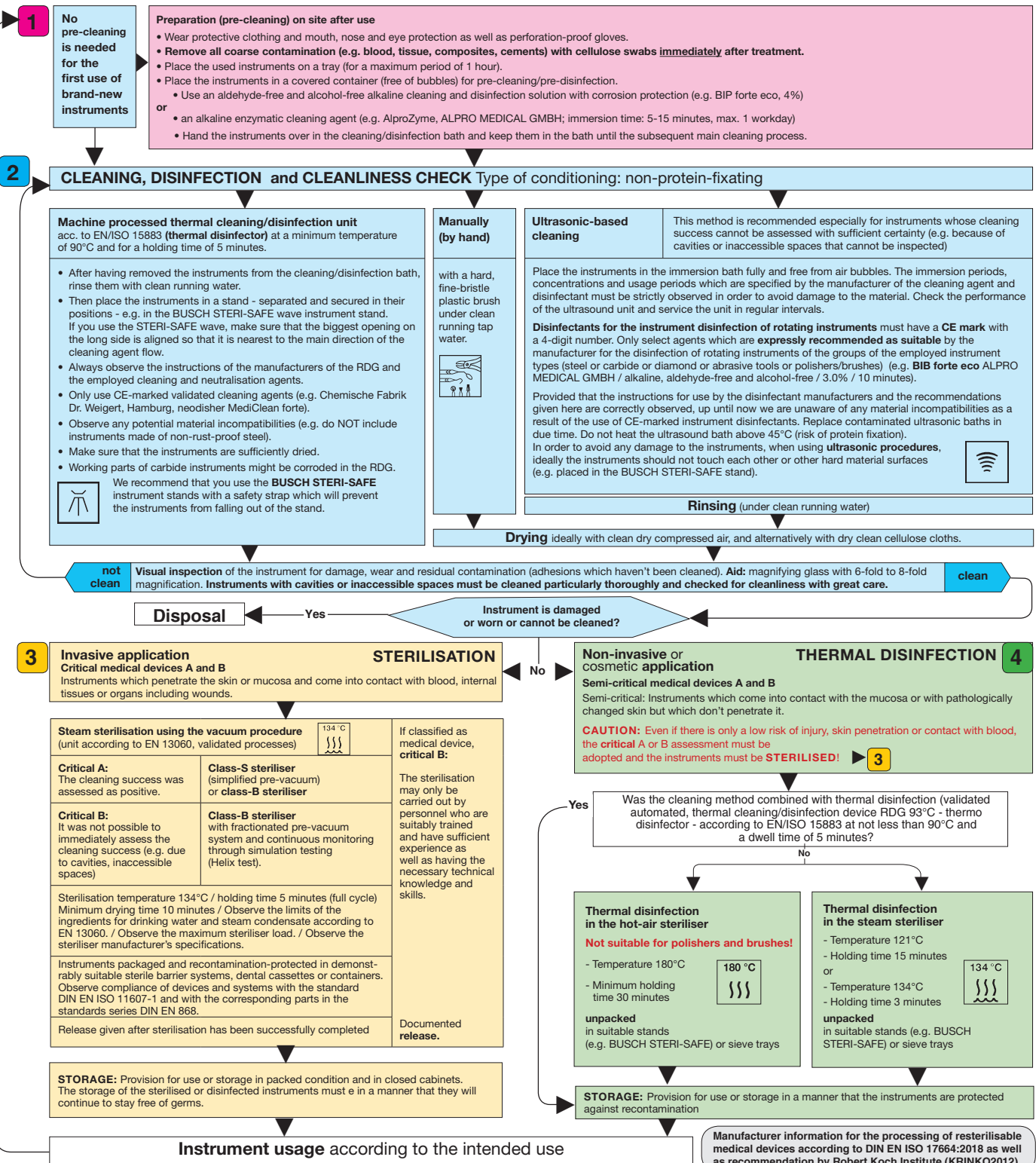
1 Preparation 2 Cleaning 3 Sterilisation 4 Thermal disinfection

Area of application: Rotating instruments made of steel, carbide, ceramic or diamond as well as polishers, abrasive tools and brushes that are intended for the application on humans.

The instruments are delivered non-sterile. Before they are used for the first time as well as after each use, they must be disinfected or cleaned and disinfected or, if required, sterilised. Grinding caps and cap mandrels have to be disinfected and sterilized unmounted for hygienic and technical reasons.

Restrictions for the processing of resterilisable instruments: Brushes for prophylaxis are single-use products because it cannot be guaranteed that the brushes can be cleaned entirely free from residue.

For non-rust-free instruments, disinfectants and cleaning agents with corrosion protection must be used. **Non-rust-free instruments are not suitable for steam sterilisation.** The product service life is determined by wear and damage caused by the instruments' use. Therefore, specific details regarding the number of processing runs cannot be provided. Never use hydrogen peroxide (H₂O₂) for disinfection of the instruments because the risk of damage to the material cannot be excluded.

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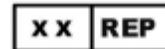
- select appropriate qualified cleaners/ disinfectants giving special regard to material friendliness; do not use any household detergents; do not use acids or alkaline solutions
- use consistent dosing (no overdosing) according to the manufacturer's specifications; take evaporation into account when using cleaner/ disinfectant baths for several days (not only water evaporates, the concentration of chemical ingredients will increase), ready-to-use solutions are preferable compared to concentrates
- comply with the maximum permissible dwell times in cleaners/disinfectants according to the manufacturer's specifications

Failure to observe this warning may result in corrosion of the instruments, which will affect their performance, stability and surface texture requiring them to be discarded prematurely.

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Our current catalogue
„Rotary instruments for
podology/footcare and
naildesign“ you can find
at www.busch.eu

Busch
There is no substitute for quality

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Rotary instruments
for podology/footcare
and naildesign

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BUSCH & CO. GmbH