
Operator's Manual should be consulted before using this autoclave.

## BASIC OPERATION

### 1 Power unit ON.

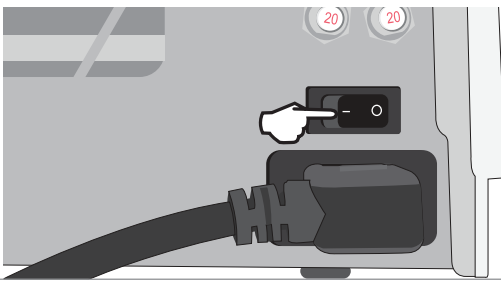
### 2 Fill with demineralised/distilled water.


NEVER USE TAP WATER


### 3 Load configurations.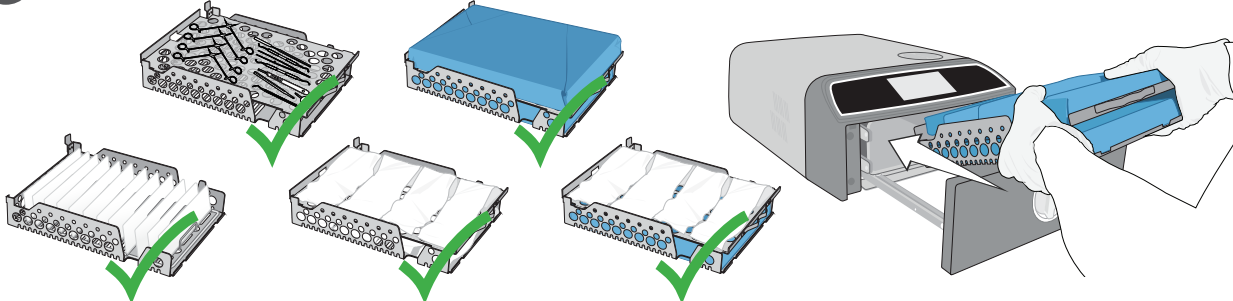


### 4 Close unit.

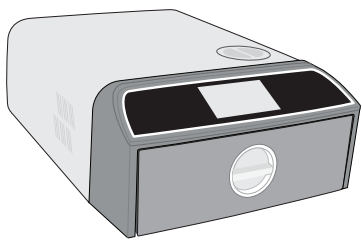
### 5 Run cycle.

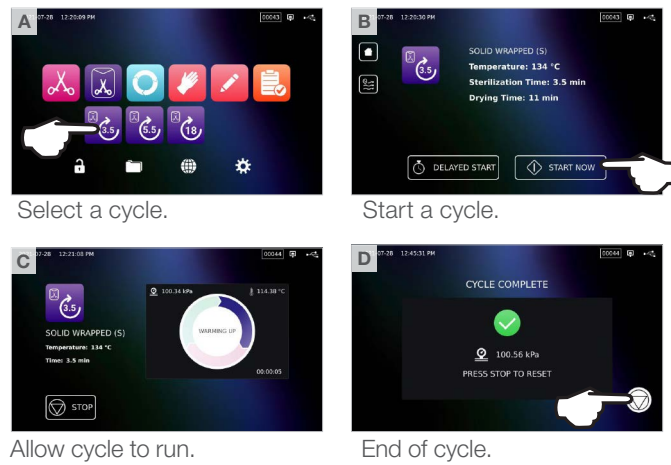
### 6 Open unit.
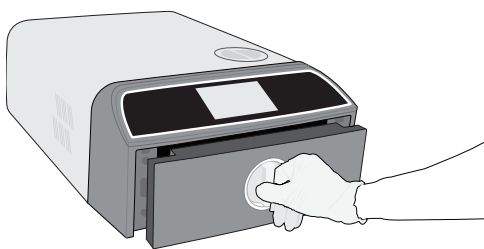
### 7 Unload instruments.

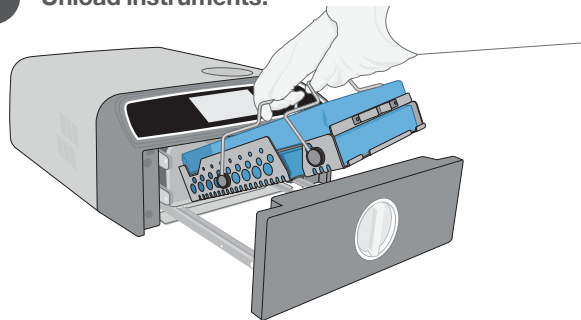
## WARM-UP MODE

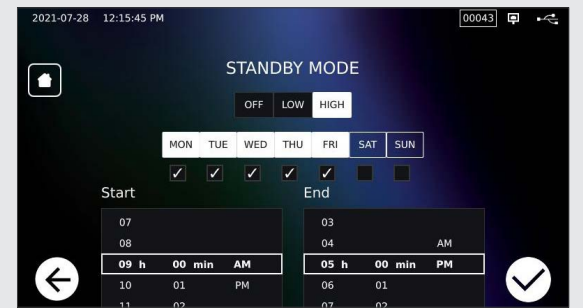
- ▶ **STAND-BY LOW** – For low to average use: 70°C / 158°F
- ▶ **STAND-BY HIGH** – For high use: 120°C / 248°F
- ▶ **STAND-BY OFF (default)** – For infrequent use. The wait time will be longer (up to 12 minutes from a cold start)

## INSTRUCTIONAL VIDEOS
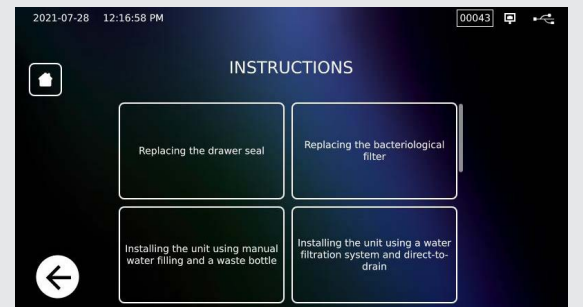
## LED COLOUR
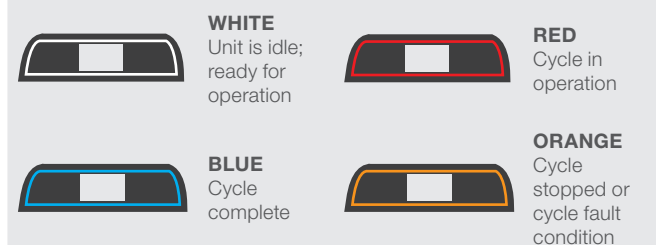
## LOCK ICON

## Spare Parts and Accessories

01-116292S	Air Filter
01-116293S	Drawer Seal, 6L
01-116294S	Tray, 6L
01-116296S	Exhaust Tube
01-116297S	Extended Exhaust Tube
01-116298S	Direct-to-Drain
01-116299S	Water Filling
01-116300S	Waste Bottle
01-116301S	USB Memory Stick
01-116302S	Water Reservoir Cap
SCI-BDSK134V	Bowie-Dick Test Kit
97902001	Helix Test Kit with 100 Chemical Indicators
97902002	Helix Text Kit with 400 Chemical Indicators
01-110282S	Power Cord EU 16A/250V
01-110285S	Power Cord UK IND plug 16A/250V
01-116496	StatMat Silicone Mat STATIM B
01-116497S	Reservoir Chamber Filter
01-116498S	Pouch Rack Kit

## IMPORTANT!


Users of this device must be sure to select the most appropriate sterilisation cycle according to the recommendations of the leading infection control authorities and local regulatory guidelines / recommendations.

## TOUCHSCREEN

**Solid / Unwrapped:**  
IUSS\* cycle for unwrapped solid instruments

**Solid / Wrapped:**  
Wrapped Solid, non-lumened instruments

**Universal:**  
Wrapped and unwrapped solid and hollow instruments, cassettes, textiles

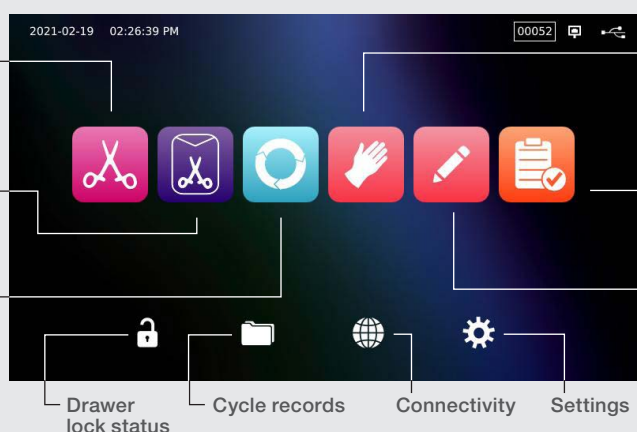**Rubber & Plastic:**  
Unwrapped and wrapped solid or hollow instruments constructed of metal, rubber and plastic

**Helix/BD Test**  
**Vacuum Test**

**Custom:**  
For devices that require different cycle parameters than those available. Set according to the instrument manufacturer's instructions.

\*Immediate Use Steam Sterilization

## MAINTENANCE SCHEDULE

<b>Daily</b>	Wipe the <b>drawer seal</b> .	<b>Monthly</b> or <b>160 Cycles</b>	Inspect the <b>water reservoir filter</b> (clean or replace).
	Clean <b>external surfaces</b> .		Clean the <b>external fill water tank</b> .
<b>Weekly</b>	Clean the <b>chamber and trays</b> .	<b>Every 6 Months</b> or <b>1,000 Cycles</b>	Perform all monthly cleaning tasks.
	Drain <b>water reservoir</b> .		Replace <b>bacteriological air filter</b> .
	Disinfect <b>external surfaces</b> .		Replace <b>drawer seal</b> .
<b>Every 2 Years</b> or <b>2,000 Cycles</b>	A <b>complete maintenance</b> of the autoclave including testing of the pressure relief valve and the power failure mode (by a SciCan-approved technician) is recommended.		

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