unguisan®  
BLUELIGHT

# KOMPLETT-SET

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## PASSIVE NAIL-CORRECTION WITH UNGUISAN® HARD COMPOSITE GEL/ NAIL-STABILISATION

Passive nail correction grounds on the observation, that involuted nails commonly show a lesser degree of curvature at the proximal end in comparison to the distal end of the nailplate.

By conserving this flatter shape of the nail while it is growing forwards, the entire nailplate will become flatter over the course of time.

This is achieved by applying a strip of **UNGUISAN® HARD COMPOSITE GEL** transverse to the direction of nail growth at the proximal end of the nail plate (see picture 1).

The extremely hard strip of composite gel now acts as a kind of stabilizing corset and will grow forwards on top of the nail.


This will ensure that the nailplate maintains its flatter curvature. Because no active force is applied, the transformation takes place without pain, free of tension and as careful as possible. Hence, this method is even applicable in cases of brittle nails, onycholysis and even suitable for at-risk-patients like diabetics or patients suffering from PVD (peripheral vascular disease).

**Pro-Tip:** After a successful course of active nail correction having utilized braces like Ross Fraser, 3TO or similar, nail stabilization with UNGUISAN® HARD COMPOSITE GEL will maintain the result and prolong the benefit for the patient.

# APPLICATION


## 1. Preparation of nail-plate:

- Thickened nails should be carefully thinned to a normal degree
- Residues of nail varnish or similar should be removed
- Gently roughen the surface of the nail
- Ensure that the nail is dry, clean and without grease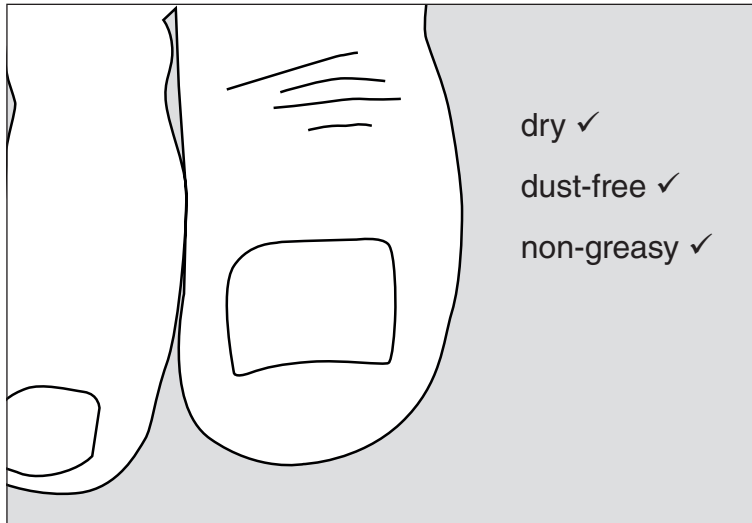

## 2. Application of UNGUISAN® PRIMER:

- Apply UNGUISAN® PRIMER with a brush (single use) on the nail plate.
- Avoid skin contact.
- The UNGUISAN® PRIMER is light-curing. The curing time with the UNGUISAN® Bluelight lamp is approx. 20 seconds.
- Close the bottle tightly immediately after use, since UNGUISAN® PRIMER can dry out.

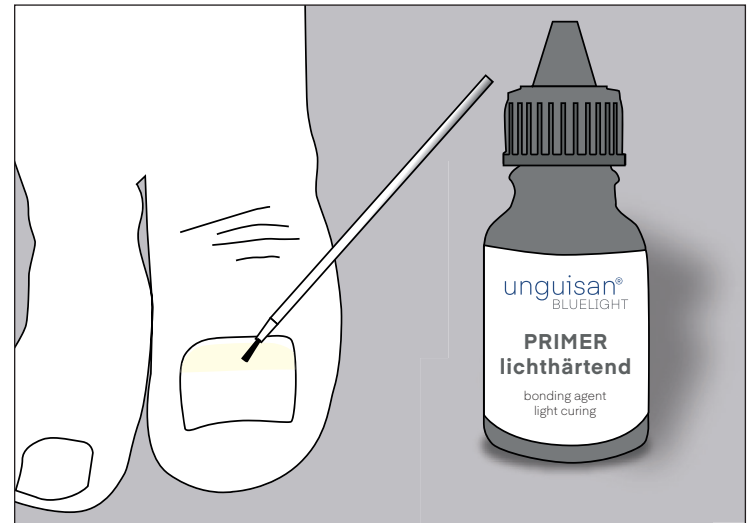
**Watch the full instructions on video!  
Simply scan the QR code and off you go!**

### 3. Application of UNGUISAN® HARD COMPOSITE GEL:


- Apply gel directly from syringe onto the nail.
- If necessary, use spatula to shape as desired.
- Recommended thickness is 2-3 mm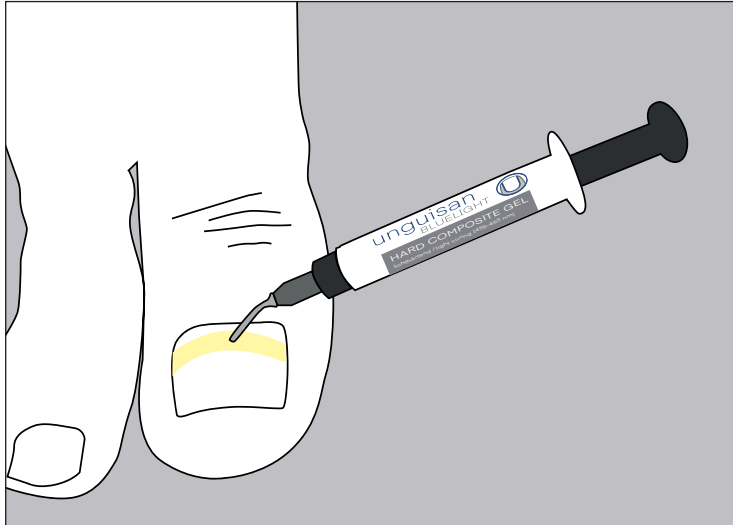

### 4. Light-curing of UNGUISAN® HARD COMPOSITE GEL:

#### **Only use UNGUISAN® BLUE LIGHT HIGH-PERFORMANCE-LED-LIGHT!!**


- From shortest possible distance expose HARD COMPOSITE GEL to light for 20 seconds
- Done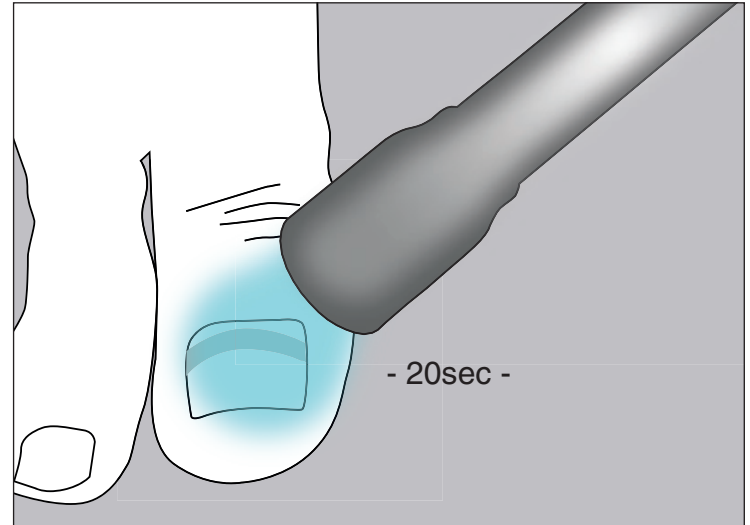


### 5. Removal or rework of UNGUISAN® HARD COMPOSITE GEL:

- Hardened composite material may be drilled using carbide burs

## FIXATION OF WIRE BRACES WITH THE UNGUISAN® BLUELIGHT STANDARD GEL

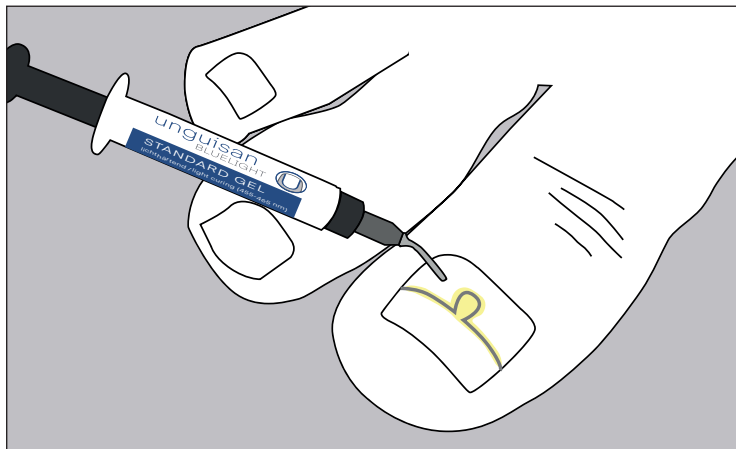
Apply UNGUISAN® Blue Light Standard Gel with dosing syringe with pinpoint accuracy.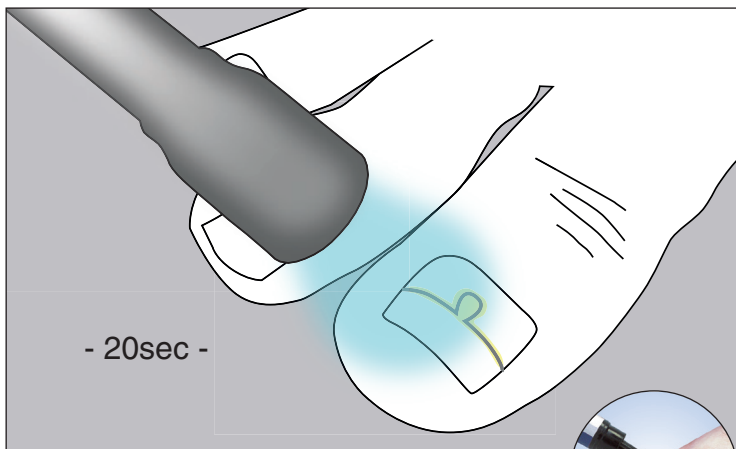


Only 20 sec. curing with the special UNGUISAN® high-power LED lamp. Ready is the quick and easy clasp fastening.

## NAIL FOLD TAMPONADE (SULCI PROTECTOR)

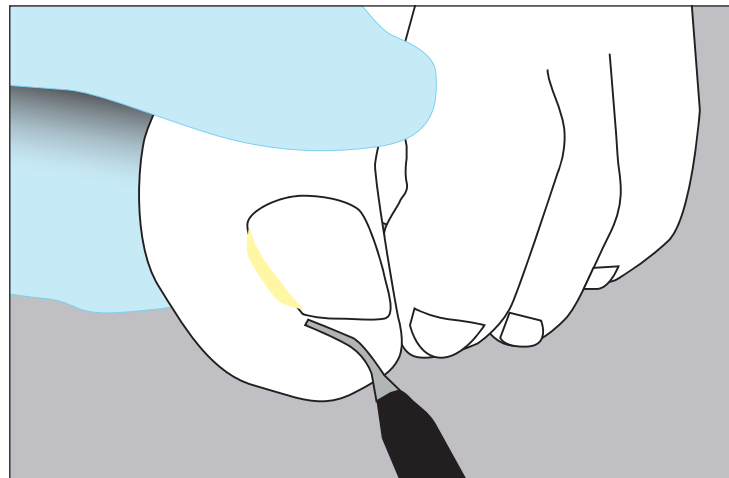Insert enough UNGUISAN® Blue Light Standard Gel into the uninjured nail fold.

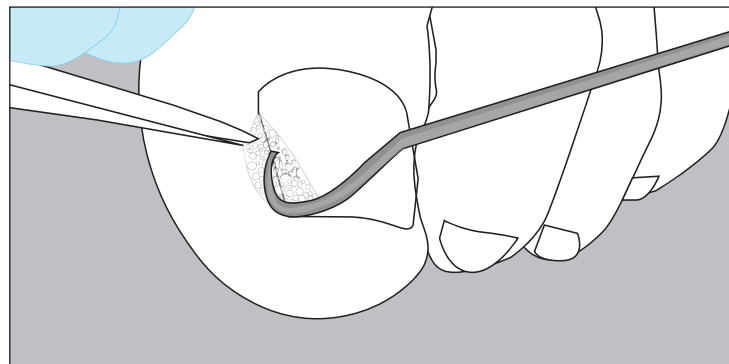Pack the nail fold with Ligasano® so that the open-pored foam soaks up the gel. If necessary, absorb excess gel with a swab. Cure the geltamponade for 20 sec. with the UNGUISAN® high-performance LED lamp.

## INHALT UNGUISAN® BLUE LIGHT SET

- UNGUISAN® Primer (Haftvermittler)
- UNGUISAN® Hard Composite Gel
- UNGUISAN® Blue Light Gel
- UNGUISAN® Blue Light Hochleistungs LED Lampe
- 5 x Einwegpinsel
- 1 x Pinselhalter

## INHALTSSTOFFE (INCIS):

### UNGUISAN® Primer:

Methacrylic acid, Triethylenglycoldimethacrylate, Tripropyleneglycolediacylate, Dipheny(2,4,6-trimethylbenzoyl) phosphine oxide

### UNGUISAN® Hard Composite Gel:

Inorganic filler, 2,2-Bis[4-(2-methacryloyloxyethoxy) phenyl]propane, TEGDMA, CI 77503, CI 77310, Silica, Silica Dimethyl Silylate, Polymer, CI 77128, 2-Hydroxy-4-methoxybenzophenon, 2,2-Dimethoxy-1,2-diphenylethan-1-one, Campherchinon, 2-Butoxyethyl-4, (dimethylamino) benzoate, CI 77891, BHT, Acetic acid, Lumilux

### UNGUISAN® Blue Light Gel Standard:

Aliphatic urethanedimethacrylate, silica, 2-Methylpropenacid, fotoinitiator, BHT, CI 12480

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## INGREDIENTS (INCIS):

### UNGUISAN® Primer:

Methacrylic acid, Triethylenglycoldimethacrylate, Tripropyleneglycolediacylate, Dipheny(2,4,6-trimethylbenzoyl) phosphine oxide

### UNGUISAN® Hard Composite Gel:

Inorganic filler, 2,2-Bis[4-(2-methacryloyloxyethoxy) phenyl]propane, TEGDMA, CI 77503, CI 77310, Silica, Silica Dimethyl Silylate, Polymer, CI 77128, 2-Hydroxy-4-methoxybenzophenon, 2,2-Dimethoxy-1,2-diphenylethan-1-one, Campherchinon, 2-Butoxyethyl-4, (dimethylamino) benzoate, CI 77891, BHT, Acetic acid, Lumilux

### UNGUISAN® Blue Light Gel Standard:

Aliphatic urethanedimethacrylate, silica, 2-Methylpropenacid, fotoinitiator, BHT, CI 12480

**Danger:** Causes severe skin burns and severe eye damage. May cause allergic skin reactions. May irritate the respiratory tract. Harmful to aquatic organisms, with longterm effects. Safety instructions: Avoid inhalation of mist/vapour/aerosol. Avoid release into the environment. Wear protective gloves / eye protection/ face protection. In contact with the skin (or hair): Take off all contaminated clothes immediately. Wash off skin with water or shower. In case of contact with the eyes: Rinse gently with water for a few minutes. If possible, remove any existing contact lenses. Continue to rinse. Call the Poison Information Centre/Doctor immediately!

For professional use only.  
Not intended for end users.

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*The UNGUISAN® anti-callus tincture softens hard skin (cornifications, onychophosis) within the nail sulcus safely and reliably. With urea and salicylic acid, hard skin quickly becomes soft and smooth again. Uncomfortable pressure, e.g. within the lateral sulcus of the nail, are alleviated. Allantoin and calendula extract support the particularly gentle care effect.*

*UNGUISAN® nail-care concentrate with its special formulation allows numerous applications. Natural ingredients like extracts from Iceland Moss and Black Currant are combined with a Silver complex. Nails sensitive to fungus are protected and it helps avoiding bacterial infections, too. Especially recommended to accompany nail prosthetics and nail correction measures, as well as a general care product for the entire nail organ.*

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*The perfect care for every skin type.*

Callusan is more than just cream foam.

Callusan is the care concept for the most common skin problems. With eight different varieties Callusan cream mousse provides healthy skin and well-being on feet, hands and body.

The Callusan Care Concept helps you and your customers to find their orientation within the product range and choose the right cream foam for their needs.

## **Greppmayr GmbH**

Hainbuchenring 4

82061 Neuried / Germany

T +49 (0) 89 759 69 69 0

F +49 (0) 89 759 69 69 69

E [info@greppmayr.de](mailto:info@greppmayr.de)

W [www.greppmayr.de](http://www.greppmayr.de)