QUICKSTART

enbio. 

CE ²²⁷⁴
ver. 30.01.2020

QUICK START

A 1 Read the operating instructions before starting work.

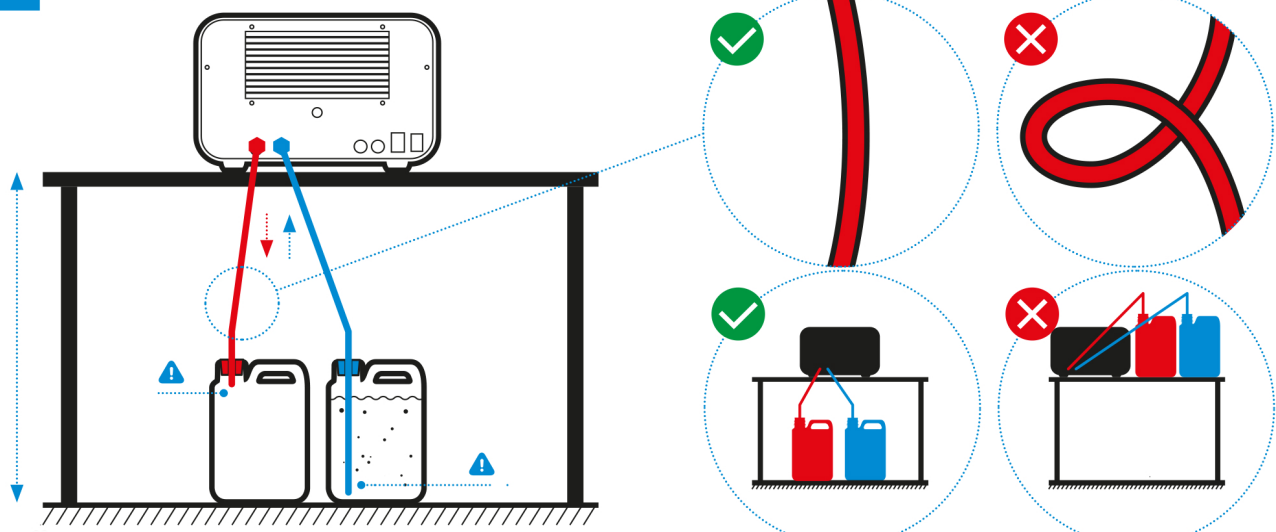
Diagram A illustrates the correct setup of the autoclave. A red tube connects the top of the autoclave to a red container, and a blue tube connects to a blue container. A vertical double-headed arrow indicates the height. Callouts show correct (green check) and incorrect (red X) ways to handle the tubes and place containers.

B

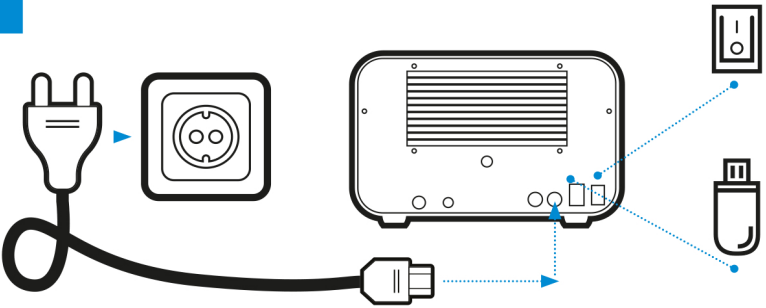
Diagram B shows the power connection. A power cord is plugged into a wall outlet and connected to the back of the autoclave. A USB stick is also plugged into the back.

C

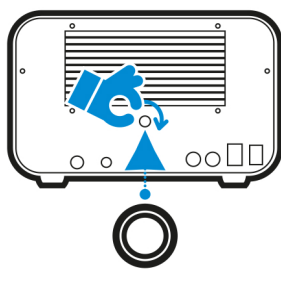
Diagram C shows the control panel. A hand is shown pressing the power button on the control panel.

D 2 The FAST 134 process is dedicated to solid, non-porous tools, and dental tools (eg scissors, handles, pliers, chisel, probes, etc.) only unpackaged, non-textile.

3 Type of tools

| | 134°C Fast | 134°C / PRION | 121°C |
|---------------|---|---|---|
| Type of tools |  |  |  |

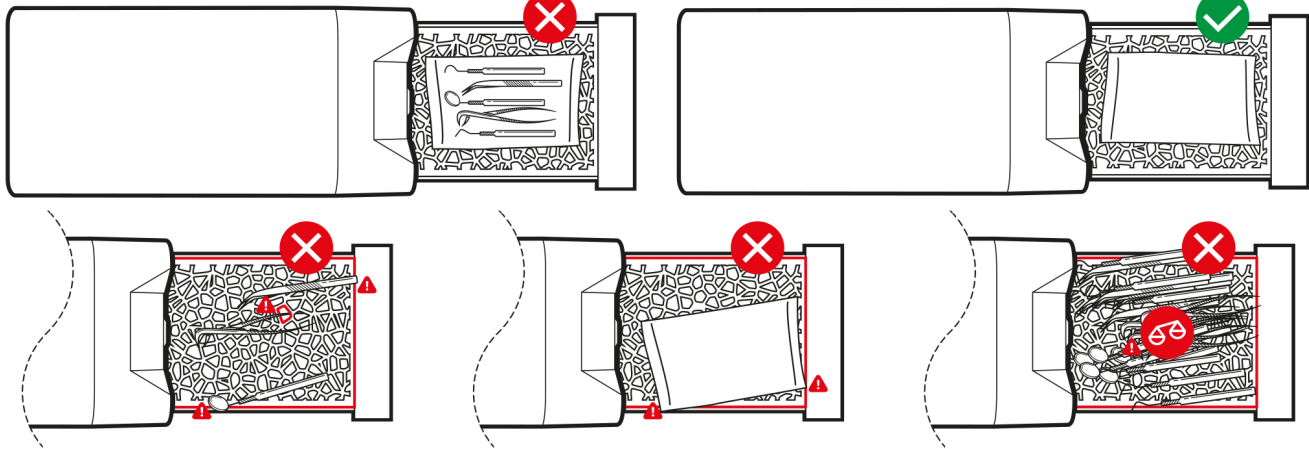
Diagram D shows four examples of tool packaging. The first two show correct (green check) ways to pack tools in a container. The last two show incorrect (red X) ways to pack tools in a container.

E

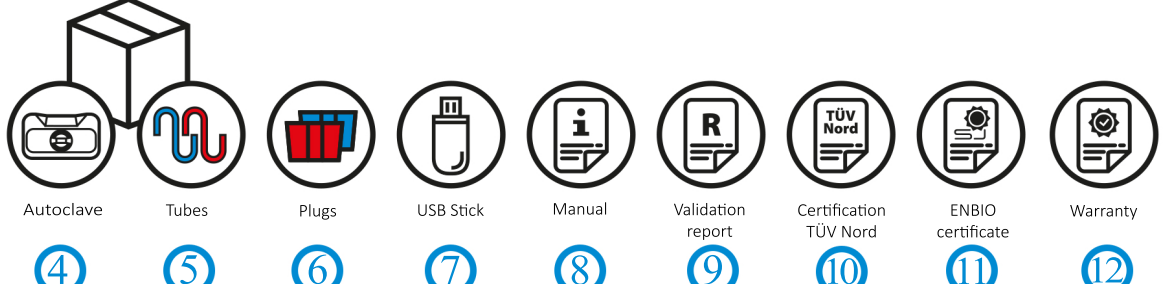
Diagram E shows a checklist of items:

- 4 Autoclave
- 5 Tubes
- 6 Plugs
- 7 USB Stick
- 8 Manual
- 9 Validation report
- 10 Certification TÜV Nord
- 11 ENBIO certificate
- 12 Warranty

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Enbio Group AG