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### **german/english/russian**

Dear customer,

Congratulations on the purchase of this foot care unit.

You have acquired a high-quality working tool which, if looked after properly, will undoubtedly give you years of service.

In designing the foot care unit, particular attention has been paid to the ergonomically optimal positioning of all operating elements in addition to timeless design.

These new models have firmly closed drawers. The penetration of cutting dust in the unit should thus be kept to a minimum.

Given the various accessories available, the foot care unit is ideal for use with the SIRIUS NT MICRO foot care spray device and the SATURN/LUNA/TRITON AT MICRO suction devices.

High-quality materials have been used in this foot care unit, thus guaranteeing long-term reliability.

Please read these instructions for use carefully before using the foot care unit.

Thank you for your co-operation.

Yours

EDUARD GERLACH GmbH

**Bäckerstraße 4-8**  
**D-32312 Lübbecke**

**☎ +495741/330-0**  
**Fax +495741/347300**

Batch: 2012/06

## **1. Objective**

The foot care unit is designed for stationary use in:

foot care and for pedicures

## **2. Technical data**

- Mains voltage: 230 V/50 Hz
- Power input: 3600 W (max.)
- Protective category: I- BF
- Dimensions (without extensions): W x D x H (in mm)  
623 x 483 x 759
- Weight: 57.1 kg

EDUARD GERLACH GmbH reserves the right to make technical and visible changes to their products at any time, without prior notice.

## **3. Scope of delivery**

- 1 Gerlach Foot Care Cabinet
- 1 Operating Instructions
- 1 Handpiece holder
- 1 Insert for Saturn/Luna
- 1 Insert for Sirius/Puck
- 3 Cutter boxes
- 3 Instrument trays
- 1 Drawer insert AT/NT B532 x T378 x H35
- 1 Multiple outlet

Please contact Eduard Gerlach GmbH immediately if any item is defective or your order incomplete.

## **4. Safety instructions**

This adhesive label on the product signifies:

**Follow the information given in the operating instructions!**

Point out the significance and the perils to all users of the product.

## **5. System allocation**

The foot care unit can be operated with the following devices:

**SIRIUS NT MICRO foot care spray device**  
**SATURN foot care suction device**  
**LUNA foot care suction device.**  
**TRITON foot care suction device**

## **6. Operation**

- Attach the hand piece support (optional) to the left or right side of the unit with the screws supplied
- Complete the hand piece support using either the spray or suction hand piece (optional) by simply inserting from above. Secure the adaptor with the screws supplied.
  
- Place the SIRIUS/SATURN/LUNA/TRITON foot care unit on the pull-out section
- Insert the main plug of the foot care unit into the 4-pin socket

### **Please proceed as follows:**

- Insert the mains plug of the multiple outlet into the wall socket.
- Switch on the electricity supply at the ON/OFF switch on the multiple outlet.

The illuminated On/Off switch indicates that the device is ready for use. The device is now connected to the mains and is ready for use.

### **Please note:**

**! The maximum connection value of the multi-pin socket is 3600 W.**

**! On safety grounds, do not connect any other multi-pin socket into the socket available.**

## **7. General safety instructions**

The foot care unit must be used only on a horizontal level. It may topple over if placed on a slanting surface (5° or 10°). Please ensure that loaded or full drawers are not pulled out completely at the same time.

Please follow the instructions in use for the device being used.

## **8. Hygiene**

If needed, the working plate can be disinfected with **Mikrobac forte**. Cleaning with a damp cloth will suffice for all other surfaces.

### **Please note:**

**! Please follow the application rates and contact times provided by the manufacturer to ensure successful cleaning.**

**! The use of another disinfectant may damage the surface. The warranty will then be void!**

**! Do not allow the disinfectant to penetrate inside the unit.**

## **9. UV – decontamination unit**

The UV sterilisation compartment serves as a storage place low in germs for pedicure instruments which have already been disinfected or sterilised. Tongs and scissors should be placed in the opening position on the tray. Avoid placing instruments on top of each other. Please place rotating instruments in the clipper stands.

### **Operation**

The UV decontamination unit is operational once the unit has been switched on at the ON/OFF switch on the multiple outlet.

The UV light is automatically extinguished when the drawer is opened.

### **Please note:**

**! The UV radiation can only provide an environment low in germs for your instruments. You must always ensure that you sterilise your instruments**

**! Place only completely dry instruments in the UV unit as otherwise the mirror could become damaged due to moisture!**

**! The UV light must be extinguished immediately on opening the drawer. If this function is defective, the UV decontamination unit must not be used!**

**! Replace the tube after approximately 2000 hours as the bulb strength will diminish.**

## **10. Maintenance**

Please follow the instructions for use for the given device.

## **11. Environmental**

Please contact the relevant regional authorities to ensure disposal in accordance with environmental requirements. In order to avoid misuse and danger, please switch off the unit at the mains prior to disposal.

Contaminated devices and instruments must be disinfected prior to disposal.

## **12. Spare parts**

Order number	Item
701875103	multiple outlet with on/off switch
302280200	UV tube, TUV 15 W
704514501	Rubber foot for instrument drawer

**! In the event of a defect, please ensure that only ORIGINAL SPARE PARTS are used.**

### **12.1 Accessories**

Order number	Item	Reference
701506503	Lampholder Universal	
317100500	Refined steel container 12 x 12 cm	Can be used alone to collect waste or with lower support. Can also be placed in instrument drawer.
317100600	Refined steel container 12 x 3 cm	
301525003	Support for refined steel container	Can be inserted in hand piece support.

## **13. Guarantee/Warranty**

EDUARD GERLACH GMBH provides a 12-month guarantee/warranty for the GERLACH foot care unit as from the date of purchase. The guarantee does not extend to parts that are subject to natural wear and tear through use. Similarly, the terms of the warranty do not cover the unit for improper or incorrect use.

## **14. Liability**

EDUARD GERLACH GMBH accepts no liability whatsoever for personal injury or damage arising from non-compliance with these instructions for use.

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